## Family health care TRE-1000

# Infrared Thermometer





#### **Features**

- Accurate and measures instantly
- 1 minute after showing the result, device is turned off automatically
- Automatic memory function, save up to 9 measured temperatures
- No probe filter need and easy to clean
- Ergonomic design for user

- LCD display shows normal, slight fever, high fever by color changes Green: Normality (35.1°C ~ 37.4°C)
- Orange: Slight fever (37.5°C ~ 38.5°C) Red: High fever (>38.5°C)
- FSTN LCD provides clear visibility from any different angle

#### How to use

• Body temperature measurement.





- 1. Place the probe to external ear canal and press the measurement button.
- 2. Keep the probe steady until the confirmation beep is heard.
  - \*When the buzzer switch is set to off, press the measurement button and hold it for approximately 2 seconds.
- 3. Confirmation beep sound is that the body temperature has been measured, and the measurement shows on Color LCD Display. If the body temperature is too low or normal, a 'beep' will sound for approximately 2 seconds with green screen. The device will sound beep 4 times when the reading temperature is slight fever or high fever.
  - \* The color LCD Display automatically turns off after 5 seconds.

### **Specifications**

Content
WI-A Healthcare Infrared Thermometer TRE-1000
Infrared detection
Body temperature Measurement. Memory Function Sound ON/OFF
LCD display
Body temperature : 34°C ~42.2°C
36°C ~ 39°C ± 0.2°C 34°C ~ 35.9°C ± 0.3°C
39.1°C ~ 42.2°C ± 0.3°C
0.1°C
15°C ~ 40°C / 15 ~ 95% RH (non condensing)
-20°C ~ 55°C / 15 ~ 95% RH (non condensing)
700 - 1060hPa
TRE-1000 Ver1.0.0
Automatic memory function, save up to 9 results
Automatic power off after 60 seconds
DC 3V, Battery(LR03) X 2, at least 1,000
161.4mm x 54.3mm x 66.7mm
84g

















Mannyeon-ro 151 74-7, Hwaseong-si, Gyeonggi-do **TEL**: 031-359-8151 **FAX**: 031-359-8152

Contact: 82-1533-3135 Email: healthcare@wi-a.com www.wi-a.com